

Morning Checks - non-Dynamic FM

Receivers & Transmitters

Questions	Test	Hearing instrument expected outcome	Transmitter expected outcome
Does hearing instrument work?	Listen to aid alone without the receiver	Sound should be heard via steto clips	NA
Does Roger/FM receiver mute hearing instrument microphone?	Connect receiver and listen to FM only (single dot) with Transmitter turned off	Should hear a buzz in the hearing instrument	NA
Does transmitter work on FM + M or Roger + M?	Switch the receiver to FM/Roger + M (two dots)	Should hear a response	Should hear a response
Final sound check	Listen to the quality of the signal	Should be clear and undistorted	Should be clear and undistorted

This sheet can be photocopied in order to keep a record of settings and daily checks for individual users.


Note

If you are experiencing intermittent problems please check the following:

- Check that the audio shoe is the correct model. This must be an MLX / MLXS shoe signified by a grey base. Check for a loose fit or damage to the electrical contacts.
- Ensure the hearing instrument battery is new. A battery tester will not check working condition therefore it may indicate a charged battery that will not operate the receiver.
- Check transmitter microphone leads are secure and the volume of the hearing instrument is optimal.
- Check that the earmolds are free from wax.
- If you suspect that the transmitter is not working go to the transmitter troubleshooting

We recommend to have a testing kit consisting of: Steto clips, listening head phones for implants, spare hearing aid batteries, spare audio shoe, cleaning wipes and a listening checker for receiver.

Trouble Shooting - Roger Touchscreen Mic

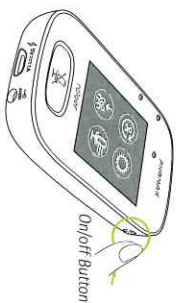

To put microphone to sleep or wake it up press On/Off button



- Please ensure The Touchscreen is fully charged.
- Quickly press the "ON" button to ensure that it is not in "Sleep Mode."
- You can also play music via a phone, or other audio device, directly into audio input socket to ensure the Touchscreen is transmitting. You can listen to the signal by using a Roger X receiver and The Roger Checker. Or via a hearing aid with a Roger receiver and steto clips.
- The display and the lights will give an indication of its current status.
- If The Touchscreen is still not responding, it can be reset by pressing the Mute Button and the ON button simultaneously for 10s and The Transmitter will reboot

Display options

Symbols

Always visible Battery status / Charging / Low battery

Remaining time for use

Microphone mode: Lanyard/Small Group/Printing

Network mode: Network/Subnet

Visible only in use Audio: Input/Output/Boom

USB cable connected

Network role: Stand-alone/Primary/Secondary

Light indication

Symbols

On-Off

Charging

Fully Charged

Switched on

Recharge device

On Off

Charging

Fully Charged

Muted

Primary microphone not available